[EN] Consumer Information

Strataderm is a rapidly drying, non-sticky, transparent silicone gel formulation for the treatment of scars, both old and new.

Strataderm can be easily applied to all areas of the skin, including exposed areas like the face and neck as well as joints and hairy areas without the need for shaving.

When used as directed Strataderm dries to form a silicone gel sheet, a protective layer that is gas permeable and waterproof which hydrates and protects scars from chemical and microbial invasion, allowing scars to normalize the collagen synthesis cycles, producing a normal mature scar.

Strataderm is used to soften and flatten scars, relieve the itching and discomfort as well as to reduce redness and discoloration associated with scars.

Strataderm was developed for the use on all types of scars including those resulting from:

- general and cosmetic surgery
- trauma
- chronic wounds
- burns
- bites
- acne
- and other diseases of the skin

The Updated International Clinical Recommendations on Scar Management confirmed that silicone gel should be used as first line therapy in the initial management of scars and in the prevention of hypertrophic scars and keloids.*

Intended use

Strataderm is used for the prevention and treatment of keloids and hypertrophic scars.

Strataderm can also be used in conjunction with other invasive scar treatment options to improve overall results, including surgical excision and resurfacing, intralesional injections and pressure garments.


Strataderm is suitable for children and people with sensitive skin.

Strataderm is safe to use during pregnancy, whilst breastfeeding and on infants.

Directions for use

On first use of 5g, 10g and 20g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50g tubes remove the cap, then the protective seal and close with the cap after use.

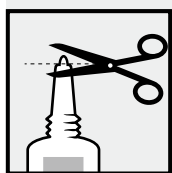Ensure that the scar is clean and dry. Apply a very thin layer of Strataderm to the scar and allow to dry. Strataderm should be applied once daily, or twice daily to exposed areas or as advised by your physician.

For best results Strataderm should maintain continuous contact with the skin (24 hours a day/7 days a week).

When applied correctly, Strataderm should be dry in 3–4 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a tissue and allow the drying process to continue.

Once dry, Strataderm can be covered with sunscreen, cosmetics, pressure garments or casts.

Recommended duration of treatment

Strataderm is recommended as a minimum treatment of 60–90 days (24 hours a day/7 days a week). Larger and older scars may take longer.

Continued use is recommended until no further improvement is seen.

How much Strataderm do I need?

- **Strataderm 5g (0.17 oz)** is enough to treat a 2–3 cm (1–2 inch) linear scar for over 90 days.
- **Strataderm 10g (0.35 oz)** is enough to treat a 5–7 cm (2–3 inch) linear scar for over 90 days.
- **Strataderm 20g (0.7 oz)** is enough to treat a 10–14 cm (4–6 inch) linear scar for over 90 days.
- **Strataderm 50g (1.75 oz)** is enough to treat an area of 6x12 cm (2.5x5 inch) for over 90 days.

Caution

- For external use only.
- Strataderm should not be applied on third degree burns or to open wounds.
- Strataderm should not be placed in contact with mucous membranes or the eyes.
- Strataderm should not be applied over other skin treatments without the advice of your physician.
- Strataderm may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- If irritation occurs, discontinue use and consult your physician.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube.

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Stratapharma AG